

**QCARE ORAL CLEANSING AND SUCTIONING SYSTEM- cetylpyridinium chloride**  
**Sage Products LLC**

*Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.*

-----

**QCare Oral Cleansing & Suctioning System Corinz™ Antiseptic Cleansing and Moisturizing Oral Rinse**

***Drug Facts***

<b><i>Active ingredient:</i></b>	<b><i>Purpose</i></b>
Corinz™ Antiseptic Cleansing and Moisturizing Oral Rinse:	
Cetylpyridinium chloride 0.05%	Antiseptic Rinse

***Uses***

- Aids in the removal of secretions and debris and helps reduce the chance of infection in minor oral irritation.

**Oropharyngeal Suction Catheter Non-sterile**

- Aids in the removal of secretions from the oropharyngeal cavity only.

***Warnings***

**Stop use and ask a doctor if:**

- Sore mouth symptoms do not improve in 7 days.
- Swelling, rash or fever develops.
- Irritation, pain or redness persists or worsens.

**Keep out of reach of children.**

If more than used for antiseptis is accidentally swallowed, get medical help or contact a Poison Control Center right away.

***Directions***

**Toothette Oral Care Suction Toothbrush**

- **Before opening**, turn package over, burst solution packet with thumbs.
- Peel lid to open.
- Attach Toothbrush to suction line.
- Clean teeth and oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Toothbrush. Reattach Covered Yankauer to suction line.
  
- Use up to 4 times daily or as directed by a dentist or doctor.
- Children under 12 years of age: supervise use.
- Children under 3 years of age: consult a dentist or doctor.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Ensure foam is intact after use. If not, remove any particles from oral cavity.

### **Toothette Oral Care Suction Swab**

- **Before opening**, turn package over, burst solution packet with thumbs.
- Peel lid to open.
- Attach Suction Swab to suction line.
- Clean teeth and oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Swab. Reattach Covered Yankauer to suction line.
  
- Use up to 4 times daily or as directed by a dentist or doctor.
- Children under 12 years of age: supervise use.
- Children under 3 years of age: consult a dentist or doctor.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Ensure foam is intact after use. If not, remove any particles from oral cavity.

### **Oropharyngeal Suction Catheter Non-sterile**

- Peel lid to open.
- Attach Suction Catheter to suction line.
- Suction secretions from the oropharyngeal cavity.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Catheter. Reattach Covered Yankauer to suction line.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.

### ***Inactive ingredients***

Water, glycerin, xylitol, spearmint flavor, potassium sorbate, polysorbate 20, polysorbate 80, hydroxyethylcellulose, citric acid, sodium saccharin, menthol.

### ***Questions?***

Call toll-free 800-323-2220.

LATEX FREE FOR SINGLE USE ONLY MADE IN U.S.A.

Corinz Label

**Corinz™**  
Antiseptic Cleansing & Moisturizing Oral Rinse  
Active Ingredient:  
Cetylpyridinium  
chloride 0.05%

Squeeze Here

To Release Fluid

**BURST PACKET  
BEFORE OPENING  
OUTER PACKAGE**

Do not remove  
from package  
0.25 fl. oz./7 mL

R5439

107

QCARE ORAL CLEANSING AND SUCTIONING SYSTEM

# SAGE<sup>®</sup>

PRODUCTS

## Q-Care<sup>™</sup>

Oral Cleansing & Suctioning System



### Setup Instructions: Multi-Port Canister

**1** Connect tubing to suction port on canister. If additional port is needed, attach Y-Connector to suction port (see below).



**2** Connect the Covered Yankauer to tubing.

**3** Connecting tool: Detach the Covered Yankauer from tubing, attach cleansing and suctioning tool directly to tubing.

Cleansing and Suctioning Tool attach directly to standard suction lines.

Using the Covered Yankauer to aid with removal of secretions:

- Follow hospital protocol.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot cooperate voluntarily.
- Retract device until no bubbles in saliva.
- To suction, move switch to ON. When finished, move switch to OFF to prevent loss of suction power.
- Rinse after use and tubing is dry.
- Between uses, pull device over tip to clean.
- Discard Yankauer and Y-Connector if used after using for 24 hours, or according to hospital protocol.
- For single patient oral care.

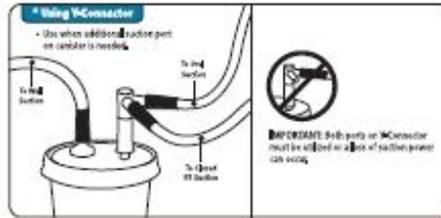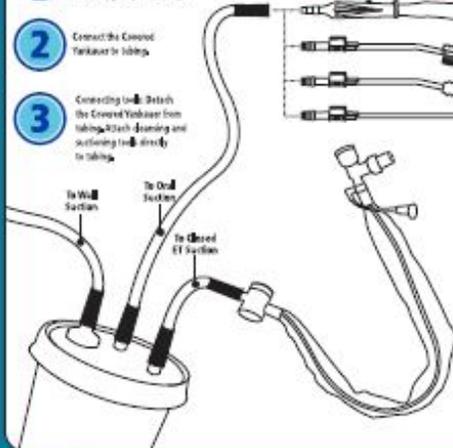

#### Contents:

- 1 Covered Yankauer and Y-Connector
- 2 Suction Tool/Rinse packages with with Corinz<sup>™</sup> Antiseptic Cleansing and Moisturizing Oral Rinse
- 3 Suction Oral Care package with with Corinz<sup>™</sup> Antiseptic Cleansing and Moisturizing Oral Rinse
- 4 Oropharyngeal Suction Collector package

#### Ingredients:

Corinz<sup>™</sup> Antiseptic Cleansing and Moisturizing Oral Rinse Active Ingredients:  
 Chlorhexidine Digluconate 0.12%  
 PEG 400 suitable sodium bicarbonate. For detailed listing of ingredients, see the enclosed individual packages.

#### Caution:

Follow SAGE Inc. written Risk Analysis (RA) to safe by or on the order of a physician or licensed practitioner.

Non-sterile • Latex free

Reorder # 6462



## SAGE<sup>®</sup>

PRODUCTS

Sage Products LLC  
 3909 Three Oaks Road | Cary, NC 27513  
 www.sageproducts.com | 800-323-3220

MADE IN USA  
 Patent: www.sageproducts.com/patents



## Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:53462-462-60	1 in 1 KIT; Type 4: Device Coated/Impregnated/Otherwise Combined with Drug	01/11/2016	

## Quantity of Parts

Part #	Package Quantity	Total Product Quantity
Part 1		1
Part 2	12 POUCH	84 mL in 12

## Part 1 of 2

### SODIUM BICARBONATE

other oral hygiene products powder

## Product Information

Route of Administration BUCCAL

## Other Ingredients

Ingredient Kind	Ingredient Name	Quantity
INGR	SODIUM BICARBONATE (UNII: 8MDF5V39QO)	
INGR	CARBOXYMETHYLCELLULOSE SODIUM (UNII: K679OBS311)	
INGR	SODIUM LAURYL SULFATE (UNII: 368GB5141J)	
INGR	SODIUM BENZOATE (UNII: OJ245FE5EU)	
INGR	WATER (UNII: 059QF0K00R)	
INGR	SACCHARIN SODIUM (UNII: SB8ZUX40TY)	

## Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
Cosmetic			

## Part 2 of 2

### CORINZ

cetylpyridinium chloride rinse

## Product Information

<b>Item Code (Source)</b>	NDC:53462-375
<b>Route of Administration</b>	ORAL

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
CETYL PYRIDINIUM CHLORIDE (UNII: D9OM4SK49P) (CETYL PYRIDINIUM - UNII: CUB7J10JV3)	CETYL PYRIDINIUM CHLORIDE	.5 mg in 1 mL

### Inactive Ingredients

Ingredient Name	Strength
WATER (UNII: 059QF0KO0R)	
GLYCERIN (UNII: PDC6A3C0OX)	
XYLITOL (UNII: VCQ006KQ1E)	
POTASSIUM SORBATE (UNII: 1VPU26JZZ4)	
POLYSORBATE 20 (UNII: 7T1F30V5YH)	
POLYSORBATE 80 (UNII: 6OZP39ZG8H)	
HYDROXYETHYL CELLULOSE (4000 MPAS AT 1%) (UNII: ZYD53NBL45)	
CITRIC ACID MONOHYDRATE (UNII: 2968PHW8QP)	
SACCHARIN SODIUM (UNII: SB8ZUX40TY)	
MENTHOL, UNSPECIFIED FORM (UNII: L7T10EIP3A)	

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		12 in 1 PACKET		
1		7 mL in 1 POUCH; Type 0: Not a Combination Product		

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	01/11/2016	

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	01/11/2016	

## QCARE ORAL CLEANSING AND SUCTIONING SYSTEM

cetylpyridinium chloride kit

### Product Information

<b>Product Type</b>	HUMAN OTC DRUG	<b>Item Code (Source)</b>	NDC:53462-464
---------------------	----------------	---------------------------	---------------

## Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:53462-464-60	1 in 1 KIT; Type 4: Device Coated/Impregnated/Otherwise Combined with Drug	01/11/2016	

## Quantity of Parts

Part #	Package Quantity	Total Product Quantity
Part 1		1
Part 2	6 POUCH	42 mL in 6

## Part 1 of 2

### SODIUM BICARBONATE

other oral hygiene products powder

## Product Information

Route of Administration BUCCAL

## Other Ingredients

Ingredient Kind	Ingredient Name	Quantity
INGR	SODIUM BICARBONATE (UNII: 8MDF5V39QO)	
INGR	CARBOXYMETHYLCELLULOSE SODIUM (UNII: K679OBS311)	
INGR	SODIUM LAURYL SULFATE (UNII: 368GB5141J)	
INGR	SODIUM BENZOATE (UNII: OJ245FE5EU)	
INGR	WATER (UNII: 059QF0KO0R)	
INGR	SACCHARIN SODIUM (UNII: SB8ZUX40TY)	

## Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
Cosmetic			

## Part 2 of 2

### CORINZ

cetylpyridinium chloride rinse

## Product Information

<b>Item Code (Source)</b>	NDC:53462-375
<b>Route of Administration</b>	ORAL

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
CETYL PYRIDINIUM CHLORIDE (UNII: D9OM4SK49P) (CETYL PYRIDINIUM - UNII: CUB7J10JV3)	CETYL PYRIDINIUM CHLORIDE	.5 mg in 1 mL

### Inactive Ingredients

Ingredient Name	Strength
WATER (UNII: 059QF0KO0R)	
GLYCERIN (UNII: PDC6A3C0OX)	
XYLITOL (UNII: VCQ006KQ1E)	
POTASSIUM SORBATE (UNII: 1VPU26JZZ4)	
POLYSORBATE 20 (UNII: 7T1F30V5YH)	
POLYSORBATE 80 (UNII: 6OZP39ZG8H)	
HYDROXYETHYL CELLULOSE (4000 MPAS AT 1%) (UNII: ZYD53NBL45)	
CITRIC ACID MONOHYDRATE (UNII: 2968PHW8QP)	
SACCHARIN SODIUM (UNII: SB8ZUX40TY)	
MENTHOL, UNSPECIFIED FORM (UNII: L7T10EIP3A)	

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		6 in 1 PACKET		
1		7 mL in 1 POUCH; Type 0: Not a Combination Product		

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	01/11/2016	

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	01/11/2016	

**Labeler** - Sage Products LLC (054326178)

**Registrant** - Sage Products LLC (054326178)

### Establishment

Name	Address	ID/FEI	Business Operations
------	---------	--------	---------------------

