

DREAM EYE - poloxamer 407 liquid
K&J.C Co., Ltd

Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, [click here](#).

Drug Facts

Active Ingredient: Poloxamer 407

STEP 1:

Place 3 drops of Dream Eye on each side of contact lens and rub for 20 seconds. No necessary. separate daily cleaner

STEP 2:

Remove surface debris by rinsing thoroughly with Dream Eye

STEP 3:

Place cleaned contact lens in the lens case and fill with fresh Dream Eye MPS. Soak at least four(4) hours.

Cleans, loosens and removes accumulations of film, deposits and debris from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinse, stores and rewets lenses before insertion. For dissolving of Enzymatic Contact Lens Cleaner tablets for removal of protein deposits.

PRECAUTIONS:

- ☐ Never re-use this solution.
- ☐ Keep the bottle tightly closed when not in use.
- ☐ Store at room temperature.
- ☐ Use before the expiration date marked on the bottle and carton.

ACTIONS: Cleans, loosens and removes accumulations of film, deposits and debris from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinse, stores and rewets lenses before insertion. For dissolving of Enzymatic Contact Lens Cleaner tablets for removal of protein deposits.

Keep out of the reach of children

INDICATIONS(Uses): Dream Eye is indicated for use in daily cleaning, dissolving of Enzymatic Contact Lens Cleaner tablets, rinsing and storage of daily and extended wear soft contact lenses and chemical (not heat) disinfection, as recommended by your eye care professional.

WARNING: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use

and care of your lenses and lens care products, including the lens case.

☐ To avoid contamination, do not touch tip of container to any surface. Replace cap after using.

☐ Not intended for use directly in the eye.

☐ Not for use with heat(thermal) disinfection.

DIRECTIONS:

☐ After removing the lens, place it in the palm of your hand, and place three drops of Daily Cleaner on each lens surface. or Soak at least four(4) hours in Dream Eye.

☐ Rub the lens for 20 seconds either in the palm of your hand using the forefinger of the other hand, or between the thumb and forefinger. Be sure to clean each side of the lens.

☐ Rinse the lens thoroughly by holding the lens in a steady stream of saline solution for approximately 10 seconds.

☐ Disinfect using the procedure which your practitioner has recommended.

Inactive ingredient: Sodium Chloride, Sodium Borate, Boric Acid, HYPROMELLOSES, EDETATE DISODIUM, Water

가로:56mm * 세로:125mm * 깊이:40mm



DREAM EYE

poloxamer 407 liquid

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:52345-3001
Route of Administration	OPHTHALMIC		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
POLOXAMER 407 (UNII: TUF2IVW3M2) (POLOXAMER 407 - UNII:TUF2IVW3M2)	POLOXAMER 407	0.1 mL in 1 mL

Inactive Ingredients

Ingredient Name	Strength
SODIUM CHLORIDE (UNII: 451W47IQ8X)	
SODIUM BORATE (UNII: 91MBZ8H3QO)	
BORIC ACID (UNII: R57ZHV85D4)	
HYPROMELLOSES (UNII: 3NXW29V3WO)	
EDETATE DISODIUM (UNII: 7FLD91C86K)	
WATER (UNII: 059QF0KO0R)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:52345-3001-1	40 mL in 1 BOTTLE, PLASTIC		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
unapproved drug other		10/13/2010	

Labeler - K&J.C Co., Ltd (690257639)

Registrant - K&J.C Co., Ltd (690257639)

Establishment

Name	Address	ID/FEI	Business Operations
K&J.C Co., Ltd		690257639	manufacture

Revised: 10/2010

K&J.C Co., Ltd