

A2492 SINGLE SHOT EPIDURAL - regional anesthesia kit
Smiths Medical ASD, Inc.

APLICARE POVIDONE-IODINE SOLUTION (povidone-iodine solution) solution
[Aplicare, Inc.]

3/4 Ounce Povidone Iodine Packet

Povidone-iodine 10%

Antiseptic

Warnings

Do not use

- if allergic to iodine
- in the eyes

For external use only

Ask a doctor before use if injuries are

- deep or puncture wounds
- serious burns

Stop use and ask a doctor if

- redness, irritation, swelling or pain persists or increases
- infection occurs

Avoid pooling beneath patient

Keep out of reach of children. In case of accidental ingestion, seek professional assistance or consult a poison control center immediately.

Package Label Display Panel

NDC 52380-0001-3
DIN 02076144

TEAR HERE
POUR



3/4 FLUID OUNCE
POVIDONE-IODINE
SOLUTION

0.75 Fl. oz. (22.5 mL)

STERILE unless opened or damaged.

Drug Facts	
Active ingredient	Purpose
Povidone-iodine USP 10%	Antiseptic
Use antiseptic skin preparation	
Warnings	
Do not use if allergic to iodine	
For external use only	

Reorder No. L-3001

A 7K022 EXP 10-10


TEAR HERE
POUR

Drug Facts (continued)
Warnings
Ask a doctor before use if injuries are ■ deep or puncture wounds ■ serious burns
Stop use and ask a doctor if
■ redness, irritation, swelling or pain persists or increases ■ infection occurs
Do not use in eyes
Avoid pooling beneath patient
Keep out of reach of children. In case of accidental ingestion, seek professional assistance or consult a poison control center immediately.
Directions apply locally as needed
Other information
■ 1% titratable iodine ■ latex free ■ for hospital or professional use only
Inactive ingredients citric acid, disodium phosphate, nonoxynol-9, sodium hydroxide, water
Questions or comments?
☎ 1-800-760-3236 (Mon to Fri 8:30 AM-5:00 PM EST)




APLICARE, INC.
MERIDEN, CT 06450
U.S.A.
DOLLARD-des-ORMEAUX,
QUEBEC H9G 2S3
www.aplicare.com
0407

Package Label Display Panel



REF A2492
SINGLE SHOT EPIDURAL



PROCEDURAL COMPONENTS:

- 1 Ultra Sound Gel, 20 grams
- 2 Plastic Syringes (20mL, Luer Slip)
- 1 Plastic Syringe (3mL, Luer Slip)
- 1 Fenestrated Drape

PREP COMPONENTS:

- 4 Gauze Sponges
- 3 Sponge Applicators
- 1 Povidone-Iodine Solution, ¼ oz.

WARNING(S):


- A needle stick with a contaminated needle may cause infectious disease.

PRECAUTIONS:


- **Use Aseptic technique.**
- To help prevent needle-stick injuries, needles should not be recapped or purposely bent. If excessive resistance is met during needle insertion, do not force the needle as damage may occur. To help avoid needle breakage, do not attempt to straighten a bent needle; discard it and complete the procedure with a replacement needle.
- After use, place sharps in a suitable sharps container. Dispose of contaminated product in a safe manner according to Centers for Disease Control and Prevention (USA) and Federal/State/Local regulations (EPA, OSHA) and health care facility guidelines or local equivalent.
- Do NOT Resternalize.
- To be used only by individuals familiar with single shot epidural procedures. For specific techniques and procedures, refer to standard textbooks.

STORE AT CONTROLLED ROOM TEMPERATURE

The Smiths Medical and Portex design marks are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.



(01) 0 0351688 05215 8




STERILE EO Rx ONLY

Caution • Do Not Reuse • Does not contain Natural Rubber Latex • Do not use if package is damaged • Sterilized using ethylene oxide • **Caution:** Federal (U.S.A.) law restricts this device to sale by or on the order of a physician

Smiths Medical ASD, Inc.
Keene, NH 03431 USA
Made in USA
www.smiths-medical.com

PTA2492, REV. 002 03/12

© 2012 Smiths Medical. All rights reserved.





SINGLE SHOT EPIDURAL



REF A2492



0000-00
 yyyy-mm
 L01 000000

smiths medical

A2492 SINGLE SHOT EPIDURAL				
regional anesthesia kit kit				
Product Information				
Product Type	MEDICAL DEVICE	Item Code (Source)	NHRIC:51688-9525	
Packaging				
#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:51688-9525-2	10 in 1 CASE		
1		1 in 1 PACKAGE, COMBINATION		
Quantity of Parts				
Part #	Package Quantity	Total Product Quantity		
Part 1	1 PACKET	22.5 mL		
Part 1 of 1				
APLICARE POVIDONE-IODINE				
povidone-iodine solution				
Product Information				
Item Code (Source)	NDC:52380-0001			

Route of Administration TOPICAL

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
POVIDONE-IODINE (UNII: 85H0HZU99M) (IODINE - UNII:9679TC07X4)	POVIDONE-IODINE	0.10 mg in 1 mL

Inactive Ingredients

Ingredient Name	Strength
CITRIC ACID MONOHYDRATE (UNII: 2968PHW8QP)	
SODIUM PHOSPHATE, DIBASIC (UNII: GR686LBA74)	
SODIUM HYDROXIDE (UNII: 55X04QC32I)	
NONOXYNOL-9 (UNII: 48Q180SH9T)	
WATER (UNII: 059QF0K00R)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:52380-0001-3	22.5 mL in 1 PACKET		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part333A	03/01/1984	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
premarket notification	K965017	12/14/2005	

Labeler - Smiths Medical ASD, Inc. (137835299)

Establishment

Name	Address	ID/FEI	Business Operations
Smiths Medical ASD, Inc.		137835299	relabel, manufacture

Establishment

Name	Address	ID/FEI	Business Operations
Aplicare, Inc.		107255002	manufacture