

A2528 NERVE BLOCK- anesthesia conduction kit
Smiths Medical ASD, Inc.

Drug Facts

Povidone-iodine

Antiseptic

Antiseptic skin preparation

Do not use-if allergic to iodine

-in the eyes

For external use only

Ask a doctor before use if injuries are

-deep or puncture wounds

-serious burns

Stop use and ask a doctor if-redness, irritation, swelling or pain persists or increases

-infection occurs

Avoid pooling beneath patient

Avoid excessive heat. Store at room temperature.

Keep out of reach of children

In case of accidental ingestion, seek professional assistance or consult a poison control center immediately.

Apply locally as needed.

- 1% titratable iodine
- latex free
- for hospital or professional use only

citric acid, disodium phosphate, nonoxynol-9, sodium hydroxide, water

1 800 760-3236 (Mon to Fri 8:30 AM-5:00 PM EST)

APLICARE

POVIDONE-IODINE SOLUTION

ANTISEPTIC

STERILE Solution





3/4 FLUID OUNCE
POVIDONE-IODINE
SOLUTION

0.75 Fl. oz. (22.5 mL)

STERILE unless opened or damaged.

Drug Facts

<i>Active ingredient</i>	<i>Purpose</i>
Povidone-iodine USP 10%.....	Antiseptic

Use antiseptic skin preparation

Warnings

Do not use ■ if allergic to iodine ■ in the eyes

For external use only ▶

Reorder No. L-3001

APLICARE, INC.
MERIDEN, CT 06450
U.S.A.
DOLLARD-des-ORMEAUX,
QUEBEC H9G 2S3
www.aplicare.com
0609



Questions or comments?
☎ 1 800 760-3236 (Mon to Fri 8:30 AM-5:00 PM EST)

Inactive ingredients citric acid, disodium phosphate, nonoxonyl-9, sodium hydroxide, water


■ for hospital or professional use only

5.375"


10.75"



Package Label Display Panel



REF A2528
NERVE BLOCK



PROCEDURAL COMPONENTS:

- 4 Gauze Sponges
- 1 Surgical Marker
- 1 18G x 1 1/2 in. Needle
- 1 25G x 3/8 in. Needle
- 1 Medicine Cup, 50ml
- 1 Plastic Syringe (5ml Luer Lock)
- 1 Plastic Syringe (10ml Luer Lock)

PREP COMPONENTS:

- 4 Towels
- 3 Sponge Applicators
- 1 Povidone-Iodine Solution, 1/4 oz.

WARNING(S):

- A needle stick with a contaminated needle may cause infectious disease.


PRECAUTIONS:

- Use Aseptic technique.
- To help prevent needle-stick injuries, needles should not be recapped or purposely bent. If excessive resistance is met during needle insertion, do not force the needle as damage may occur. To help avoid needle breakage, do not attempt to straighten a bent needle; discard it and complete the procedure with a replacement needle.
- After use, place sharps in a suitable sharps container. Dispose of contaminated product in a safe manner according to Centers for Disease Control and Prevention (USA) and Federal/State/Local regulations (EPA, OSHA) and health care facility guidelines or local equivalent.
- Do NOT Reuse/sterilize.
- To be used only by individuals familiar with nerve block procedures. For specific techniques and procedures, refer to standard textbooks.

Smiths Medical design mark and Portex design mark are trademarks of Smiths Medical. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.


Manufacturer:
Smiths Medical ASD, Inc.
10 Bowman Drive
Keene, NH 03431 USA
www.smiths-medical.com

MADE IN USA



STERILE EO Rx ONLY

Caution • Do Not Reuse • Not made with natural rubber latex • Do not use if package is damaged • Sterilized using ethylene oxide • Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician



(01) 2 0351688 05153 7

© 2015 Smiths Medical. All rights reserved.

smiths medical

REF A2528

NERVE BLOCK

 smiths medical
 Use 0000-00
 18G, 1999-mm
 000000
 LOT

A2528 NERVE BLOCK

anesthesia conduction kit kit

Product Information

Product Type	MEDICAL DEVICE	Item Code (Source)	NHRIC:51688-5050
---------------------	----------------	---------------------------	------------------

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:51688-5050-2	10 in 1 CASE		
1		1 in 1 PACKAGE, COMBINATION; Type 0: Not a Combination Product		

Quantity of Parts

Part #	Package Quantity	Total Product Quantity
Part 1	1 PACKET	22.5 mL

Part 1 of 1

APLICARE POVIDONE-IODINE
povidone-iodine solution

Product Information

Item Code (Source)	NDC:52380-0001
Route of Administration	TOPICAL

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
POVIDONE-IODINE (UNII: 85H0HZU99M) (IODINE - UNII:9679TC07X4)	IODINE	10 mg in 1 mL

Inactive Ingredients

Ingredient Name	Strength
CITRIC ACID MONOHYDRATE (UNII: 2968PHW8QP)	
SODIUM PHOSPHATE, DIBASIC (UNII: GR686LBA74)	
SODIUM HYDROXIDE (UNII: 55X04QC32I)	
NONOXYNOL-9 (UNII: 48Q180SH9T)	
WATER (UNII: 059QF0K00R)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:52380-0001-3	22.5 mL in 1 PACKET; Type 0: Not a Combination Product		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part333A	03/01/1984	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
premarket notification	K965017	12/14/2005	

Labeler - Smiths Medical ASD, Inc. (137835299)

Establishment

Name	Address	ID/FEI	Business Operations
Smiths Medical ASD, Inc.		137835299	manufacture

Establishment

Name	Address	ID/FEI	Business Operations
Aplicare, Inc.		107255002	manufacture

Revised: 5/2016

Smiths Medical ASD, Inc.