

**QCARE ORAL CLEANSING AND SUCTIONING SYSTEM COMPATIBLE WITH 0.12% CHLORHEXIDINE GLUCONATE (CHG) ORAL RINSE- cetylpyridinium chloride**

**Sage Products LLC**

*Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.*

-----  
**QCare Oral Cleansing & Suctioning System compatible with 0.12% Chlorhexidine Gluconate (CHG) Oral Rinse**

***Drug Facts***

**Suction Swab with Antiseptic Oral Rinse**

<b><i>Active Ingredient:</i></b>	<b><i>Purpose</i></b>
Corinz™ Antiseptic Cleansing and Moisturizing Oral Rinse: Cetylpyridinium chloride 0.05%	Antiseptic Rinse

\*Compatible for use with 0.12% Chlorhexidine Gluconate (CHG) oral rinse, tested for use up to five minutes.

NOTE: The following Uses and Directions refer to the Suction Toothbrush and Swab. For Warnings, Uses and Directions specific to the CHG rinse including use in children under 18 years of age, refer to that product's package insert and labeling.

***Uses***

**Suction Swab with Corinz Antiseptic Cleansing and Moisturizing Oral Rinse**

- Aids in the removal of secretions and debris and helps reduce the chance of infection in minor oral irritation.

**Suction Toothbrush CHG compatible\***

- Aids in the removal of dental plaque, debris and secretions.

**Oropharyngeal Suction Catheter Non-sterile**

- Aids in the removal of secretions from the oropharyngeal cavity only.

***WARNINGS***

**Stop use and ask a doctor if:**

- Sore mouth symptoms do not improve in 7 days.
- Swelling, rash or fever develops.
- Irritation, pain or redness persists or worsens.

**Keep out of reach of children.**

If more than used for antisepsis is accidentally swallowed, get medical help or contact a Poison Control Center right away.

***Directions***

**Suction Swab with Antiseptic Oral Rinse**

**Before opening**, turn package over, burst solution packet with thumbs.

- Peel lid to open.
- Remove Mouth Moisturizer and Applicator Swab.
- Attach Suction Swab to suction line.
- Clean teeth and oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Swab. Reattach Covered Yankauer to suction line.
- Place Mouth Moisturizer on Applicator Swab.
- Apply as needed to lips and inside mouth.
- Use up to 4 times daily or as directed by a dentist or doctor.
- Children under 12 years of age: supervise use.
- Children under 3 years of age: consult a dentist or doctor.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Ensure foam is intact after use. If not, remove any particles from oral cavity.

#### **Suction Toothbrush CHG compatible\***

- Peel lid to open.
- Remove Suction Toothbrush and attach to suction line.
- When using with a cleansing solution, refer to the product packaging for indications, instructions and warnings.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Toothbrush. Reattach Covered Yankauer to suction line.
- Use Swab for additional cleansing as needed.
- Use two times daily or as directed by a dentist or doctor.
- Children under 12 years of age: supervise use.
- Children under 3 years of age: consult a dentist or doctor.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Ensure foam is intact after use. If not, remove any particles from oral cavity.

#### **Oropharyngeal Suction Catheter Non-sterile**

- Peel lid to open.
- Attach Suction Catheter to suction line.
- Suction secretions from the oropharyngeal cavity.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Catheter. Reattach Covered Yankauer to suction line.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Oropharyngeal Suction Catheter Non-sterile

#### **Caution**

- Federal (U.S.A.) law restricts this device to sale by or on the order of a physician or licensed practitioner.

#### ***Inactive Ingredients***

## **Suction Swab with Corinz Antiseptic Cleansing and Moisturizing Oral Rinse**

Water, glycerin, xylitol, spearmint flavor, potassium sorbate, polysorbate 20, polysorbate 80, hydroxyethylcellulose, citric acid, sodium saccharin, menthol.

### ***Questions?***

Call toll-free 800-323-2220

NOT MADE WITH NATURAL RUBBER LATEX • FOR SINGLE USE ONLY • MADE IN U.S.A.

### **Manufactured for:**

Sage Products LLC

Cary, IL 60013

1-800-323-2220

**QCare Oral Cleansing & Suctioning System compatible with 0.12% Chlorhexidine Gluconate (CHG) Oral Rinse**

Tear Here

# SAGE<sup>®</sup> PRODUCTS

## Q-Care<sup>®</sup>

### Oral Cleansing & Suctioning System

compatible with 0.12% Chlorhexidine Gluconate (CHG) Oral Rinse



#### Setup Instructions: Multi-Port Canister

**1** Connect tubing to suction port on canister. If additional port is needed, attach Y-Connector to suction port (see below\*).


**2** Connect the Covered Yankauer to tubing.

**3** Connecting tools: Detach the Covered Yankauer from tubing. Attach cleansing and suctioning tools directly to tubing.

Cleansing and Suctioning Tools attach directly to standard suction lines.


Using the Covered Yankauer to aid with removal of secretions:

- Follow hospital protocol.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Retract sleeve until cap locks in place.
- To suction, move switch to ON. When finished, move switch to OFF to prevent loss of suction power.
- Rinse after use until tubing is clear.
- Between uses, pull sleeve over tip to cover.
- Discard Yankauer and Y-Connector (if used) after using for 24 hours, or according to hospital protocol.
- For single patient oral use.

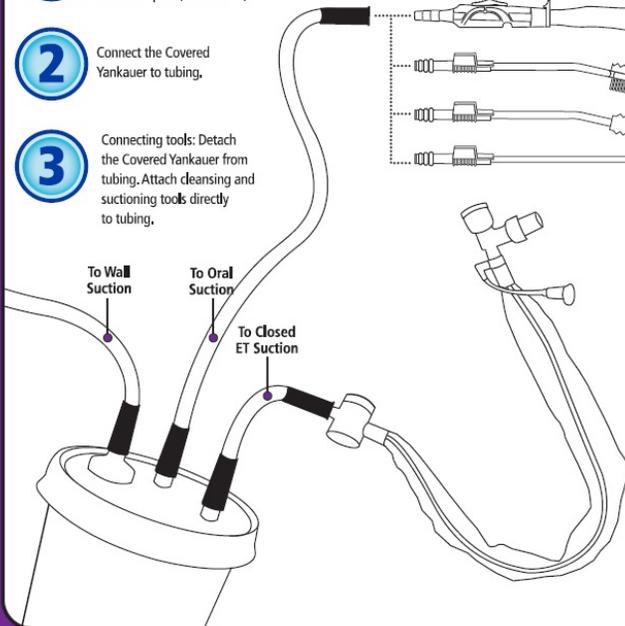

#### \* Using Y-Connector

- Use when additional suction port on canister is needed.

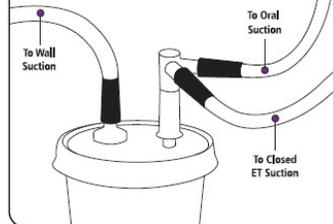

**IMPORTANT:** Both ports on Y-Connector must be utilized or a loss of suction power can occur.

#### Contents:

- 1 Covered Yankauer and Y-Connector
- 2 Untreated Suction Toothbrush packages
- 4 Suction Oral Swab packages with Corinz™ Antiseptic Cleansing and Moisturizing Oral Rinse
- 2 Oropharyngeal Suction Catheter packages

#### Ingredients:

**Corinz™ Antiseptic Cleansing and Moisturizing Oral Rinse Active Ingredient:**  
Cetylpyridinium chloride 0.05%  
Kit also contains sodium bicarbonate. For detailed listing of ingredients, see the enclosed individual packages.

#### Caution:

Federal (U.S.A.) law restricts this device (kit) to sale by or on the order of a physician or licensed practitioner.

**Non-sterile • Not made with natural rubber latex**

Reorder # 6994



# SAGE<sup>®</sup> PRODUCTS

Sage Products LLC  
3909 Three Oaks Road | Cary, Illinois 60013  
www.sageproducts.com | 800-323-2220



MADE IN U.S.A.  
Patents: sageproducts.com/patents

84748B

# SAGE<sup>®</sup> PRODUCTS

## Q-Care<sup>®</sup> Oral Cleansing & Suctioning System



Reorder #6994

DATE



TIME FOR CARE



SCAN TO TRACK



STEP 5

TIME FOR CARE

3 53462 57560 4

SCAN TO TRACK



STEP 4

TIME FOR CARE

**SAGE**<sup>®</sup>  
PRODUCTS

*Q-Care<sub>Rx</sub>*  
Oral Cleansing & Suctioning System

FEATURING  
**Corinz**<sup>™</sup>  
Antiseptic Cleansing & Moisturizing Oral Care

Reorder #6994

DATE



STEP 3

TIME FOR CARE

3 53462 57560 4

SCAN TO TRACK



STEP 2

TIME FOR CARE


3 53462 57560 4

SCAN TO TRACK



STEP 1

TIME FOR CARE

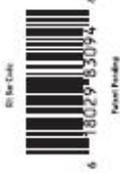

# SUCTION SWAB

With Corinz® Antiseptic Cleansing & Moisturizing Oral Rinse

CONTENTS: 1 Suction Swab with sodium Molybdate, 0.25 mg/mL Corinz Antiseptic Cleansing and Moisturizing Oral Rinse

Drug Facts	Purpose
<b>Active ingredient:</b> COMBINE ANTISEPTIC CLEANSING AND MOISTURIZING ORAL RINSE	
<b>Other ingredients:</b> Antiseptic Rinse	

**Uses:**

- Aids in the removal of secretions and debris and helps reduce the chance of infection in minor oral lesions.

**Warnings:**

Stop use and ask a doctor if:

- Sore mouth symptoms do not improve in 7 days.
- Swelling, rash or fever develops.
- Irritation, pain or redness persists or worsens.

Keep out of reach of children. More than used for antiseptic is accidentally swallowed, get medical help or contact a Poison Control Center right away.

**Directions:**

- Before opening, turn package over, burst solution sachet with thumb.
- Peel to open.
- Attach suction swab to suction line.
- Place swab in oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Swab, Flushback Covered Mouthpiece to suction line.

Use up to 4 times daily or as directed by a dentist or doctor.

Children under 12 years of age: supervise use.

Children under 3 years of age: consult a dentist or doctor.

Use a site block when performing oral care on patients with decreased levels of consciousness or those who cannot comprehend commands.

Crush foam to collect after use. Do not remove any particles from oral cavity.

**Inactive ingredients:** Water, sorbitol, xylitol, sorbic acid, potassium sorbate, potassium 20, potassium 20, hydroxyethylcellulose, citric acid, sodium saccharin, menthyl.

**Questions? Call 1-800-368-0225.**

Sage Products Inc. 10000 Sage Road, Santa Ana, CA 92705

SEE HOW TO REMOVE BOTTLE LID FOR INSTRUCTIONS



# SUCTION SWAB

With Corinz® Antiseptic Cleansing & Moisturizing Oral Rinse

CONTENTS: 1 Suction Swab with sodium Molybdate, 0.25 mg/mL Corinz Antiseptic Cleansing and Moisturizing Oral Rinse

Drug Facts	Purpose
<b>Active ingredient:</b> COMBINE ANTISEPTIC CLEANSING AND MOISTURIZING ORAL RINSE	
<b>Other ingredients:</b> Antiseptic Rinse	

**Uses:**

- Aids in the removal of secretions and debris and helps reduce the chance of infection in minor oral lesions.

**Warnings:**

Stop use and ask a doctor if:

- Sore mouth symptoms do not improve in 7 days.
- Swelling, rash or fever develops.
- Irritation, pain or redness persists or worsens.

Keep out of reach of children. More than used for antiseptic is accidentally swallowed, get medical help or contact a Poison Control Center right away.

**Directions:**

- Before opening, turn package over, burst solution sachet with thumb.
- Peel to open.
- Attach suction swab to suction line.
- Place swab in oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Swab, Flushback Covered Mouthpiece to suction line.

Use up to 4 times daily or as directed by a dentist or doctor.

Children under 12 years of age: supervise use.

Children under 3 years of age: consult a dentist or doctor.

Use a site block when performing oral care on patients with decreased levels of consciousness or those who cannot comprehend commands.

Crush foam to collect after use. Do not remove any particles from oral cavity.

**Inactive ingredients:** Water, sorbitol, xylitol, sorbic acid, potassium sorbate, potassium 20, potassium 20, hydroxyethylcellulose, citric acid, sodium saccharin, menthyl.

**Questions? Call 1-800-368-0225.**

Sage Products Inc. 10000 Sage Road, Santa Ana, CA 92705

SEE HOW TO REMOVE BOTTLE LID FOR INSTRUCTIONS



# SUCTION TOOTHBRUSH

CRG compatible\*

CONTENTS: 1 Unopened Suction Toothbrush, 1 Unopened Swab

\*Compatible for use with A,123, CRG-compatible CRG6 and CRG8, tested for use up to five minutes.

**NOTE:** The following Uses and Directions refer to the Suction Toothbrush and Swab. For Warnings, Uses and Directions specific to the CRG rinse including use in children under 14 years of age, refer to that product's package insert and labeling.

**Uses:**

- Aids in the removal of dental plaque, debris and tartar.

**Directions:**

- Peel to open.
- Remove Suction Toothbrush and attach to suction line.
- When using with a CRG6 or CRG8, refer to the product packaging for instructions, illustrations and warnings.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Toothbrush, Flushback Covered Mouthpiece to suction line.
- Use Swab for additional cleaning as needed.

Use two times daily or as directed by a dentist or doctor.

Children under 12 years of age: supervise use.

Children under 3 years of age: consult a dentist or doctor.

Use a site block when performing oral care on patients with decreased levels of consciousness or those who cannot comprehend commands.

Crush foam to collect after use. Do not remove any particles from oral cavity.

**Questions? Call 1-800-368-0225.**

Visit [www.sageproducts.com](http://www.sageproducts.com)

SEE HOW TO REMOVE BOTTLE LID FOR INSTRUCTIONS



# OROPHARYNGEAL SUCTION CATHETER

Mouthpiece

CONTENTS: 1 Suction Catheter

**Uses:**

Aids in the removal of secretions from the oropharyngeal cavity only.

**Directions:**

- Peel to open.
- Attach Suction Catheter to suction line.
- Suction secretions from the oropharyngeal cavity.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Catheter, Flushback Covered Mouthpiece to suction line.
- Use a site block when performing oral care on patients with decreased levels of consciousness or those who cannot comprehend commands.

**Caution:**

- Federal (FDA) law restricts this device to sale by or on the order of a physician or licensed professional.

SEE HOW TO REMOVE BOTTLE LID FOR INSTRUCTIONS



Corinz Label



**QCARE ORAL CLEANSING AND SUCTIONING SYSTEM COMPATIBLE WITH 0.12% CHLORHEXIDINE GLUCONATE (CHG) ORAL RINSE**

cetylpyridinium chloride kit

**Product Information**

<b>Product Type</b>	HUMAN OTC DRUG	<b>Item Code (Source)</b>	NDC:53462-994
---------------------	----------------	---------------------------	---------------

**Packaging**

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:53462-994-60	1 in 1 KIT; Type 1: Convenience Kit of Co-Package	11/16/2016	

**Quantity of Parts**

Part #	Package Quantity	Total Product Quantity
Part 1	4 PACKET	28 mL in 2
Part 2		1

**Part 1 of 2**

**CORINZ**

cetylpyridinium chloride rinse

**Product Information**

<b>Item Code (Source)</b>	NDC:53462-375
<b>Route of Administration</b>	ORAL

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
<b>CETYL PYRIDINIUM CHLORIDE</b> (UNII: D9OM4SK49P) (CETYL PYRIDINIUM - UNII: CUB7J10JV3)	CETYL PYRIDINIUM CHLORIDE	0.5 mg in 1 mL

### Inactive Ingredients

Ingredient Name	Strength
<b>WATER</b> (UNII: 059QF0KO0R)	
<b>GLYCERIN</b> (UNII: PDC6A3C0OX)	
<b>XYLITOL</b> (UNII: VCQ006KQ1E)	
<b>POTASSIUM SORBATE</b> (UNII: 1VPU26JZZ4)	
<b>POLYSORBATE 20</b> (UNII: 7T1F30V5YH)	
<b>POLYSORBATE 80</b> (UNII: 6OZP39ZG8H)	
<b>HYDROXYETHYL CELLULOSE (4000 MPAS AT 1%)</b> (UNII: ZYD53NBL45)	
<b>CITRIC ACID MONOHYDRATE</b> (UNII: 2968PHW8QP)	
<b>SACCHARIN SODIUM</b> (UNII: SB8ZUX40TY)	
<b>MENTHOL, UNSPECIFIED FORM</b> (UNII: L7T10EIP3A)	

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		4 in 1 PACKET		
1		7 mL in 1 PACKET; Type 1: Convenience Kit of Co-Package		

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	01/11/2016	

### Part 2 of 2

#### SODIUM BICARBONATE

other oral hygiene products powder

### Product Information

<b>Route of Administration</b>	BUCCAL
--------------------------------	--------

### Other Ingredients

Ingredient Kind	Ingredient Name	Quantity
-----------------	-----------------	----------

INGR	<b>SODIUM BICARBONATE</b> (UNII: 8MDF5V39QO)	
INGR	<b>CARBOXYMETHYLCELLULOSE SODIUM</b> (UNII: K679OBS311)	
INGR	<b>SODIUM LAURYL SULFATE</b> (UNII: 368GB5141J)	
INGR	<b>SODIUM BENZOATE</b> (UNII: OJ245FE5EU)	
INGR	<b>WATER</b> (UNII: 059QF0K00R)	
INGR	<b>SACCHARIN SODIUM</b> (UNII: SB8ZUX40TY)	

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
Cosmetic			

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	11/16/2016	

**Labeler** - Sage Products LLC (054326178)

**Registrant** - Sage Products LLC (054326178)

### Establishment

Name	Address	ID/FEI	Business Operations
Sage Products LLC		054326178	manufacture(53462-994, 53462-375)

Revised: 10/2019

Sage Products LLC