

MCKESSON UNNA BOOT WITH CALAMINE 4- zinc oxide and ferric oxide red dressing

MCKESSON UNNA BOOT WITH CALAMINE 3- zinc oxide and ferric oxide red dressing

McKesson Medical-Surgical

Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.

Drug Facts

Active ingredients

(in each gram)

Zinc Oxide 3%

Calamine 3%

Purpose

Skin Protectant

Skin Protectant

Uses

- Unna Boots with Zinc Oxide and Calamine helps protect skin while offering compression and support.

Warnings

For external use only.

- If condition worsens or does not improve within 7 days, consult a doctor
- Avoid contact with eyes
- Keep this and all medicines out of children's reach

Directions

- Apply as needed
- Cleanse the leg and moisturize the skin, if necessary
- If a wound is present, apply an appropriate wound dressing for moist wound healing
- Position the foot and ankle at a 90° or right angle to the leg (dorsiflexed)
- Begin wrapping the medicated bandage without tension at the base of the toes. Wrap the leg with a light tension, overlapping each layer by approximately 50%, in a spiral fashion
- Wrap up the leg stopping 1-2 inches below the posterior knee

- Apply a tubular stockinette or cohesive bandage (not included) to protect the medicated bandage and clothing

Other information

- Keep away from sunlight. Store in a dry, ventilated area
- Do not store above 77°F (25°C)

Inactive ingredients

Castor Oil, Glycerin, Methylparaben, Propylparaben.

Questions?

Call 1-800-777-4908

Principal Display Panel - Case Label

NDC 68599-6112-2

McKESSON

Unna Boots **3"**
WITH ZINC OXIDE & CALAMINE | NON-STERILE

3" x 10 yds
(7.6 cm x 9.1 m)

1
PER BOX

12
BOXES PER CASE

DO NOT REUSE

Keep away from sunlight. Store in a dry, ventilated area. Do not store above 77°F (25°C). Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

MFR # **2068**

Distributed By

McKesson Medical-Surgical Inc.
Richmond, VA 23228
Rev. 01 04/14
Made in Germany

LOT

EXP

NDC 68599-6112-2

MCKESSON

Unna Boots 3"

WITH ZINC OXIDE & CALAMINE | NON-STERILE

3" x 10 yds

(7.6 cm x 9.1 m)

1

PER BOX

12

BOXES PER CASE



DO NOT REUSE

Keep away from sunlight. Store in a dry, ventilated area. Do not store above 77°F (25°C). Not made with natural rubber latex.

Questions? Call 1-800-777-4908

■ Satisfaction Guaranteed

Distributed By
McKesson Medical-Surgical Inc.
Richmond, VA 23233
Rev 02 11/16
Made in Germany

MFR # 2068



(01)20612479192632

LOT
EXP

Principal Display Panel

NDC 68599-6112-3

McKESSON


Unna Boot **3"**
WITH ZINC OXIDE & CALAMINE
NON-STERILE

3" x 10 yds
(7.6 cm a 9.1 m)

1
PER BOX

DO NOT REUSE

MFR # **2068**

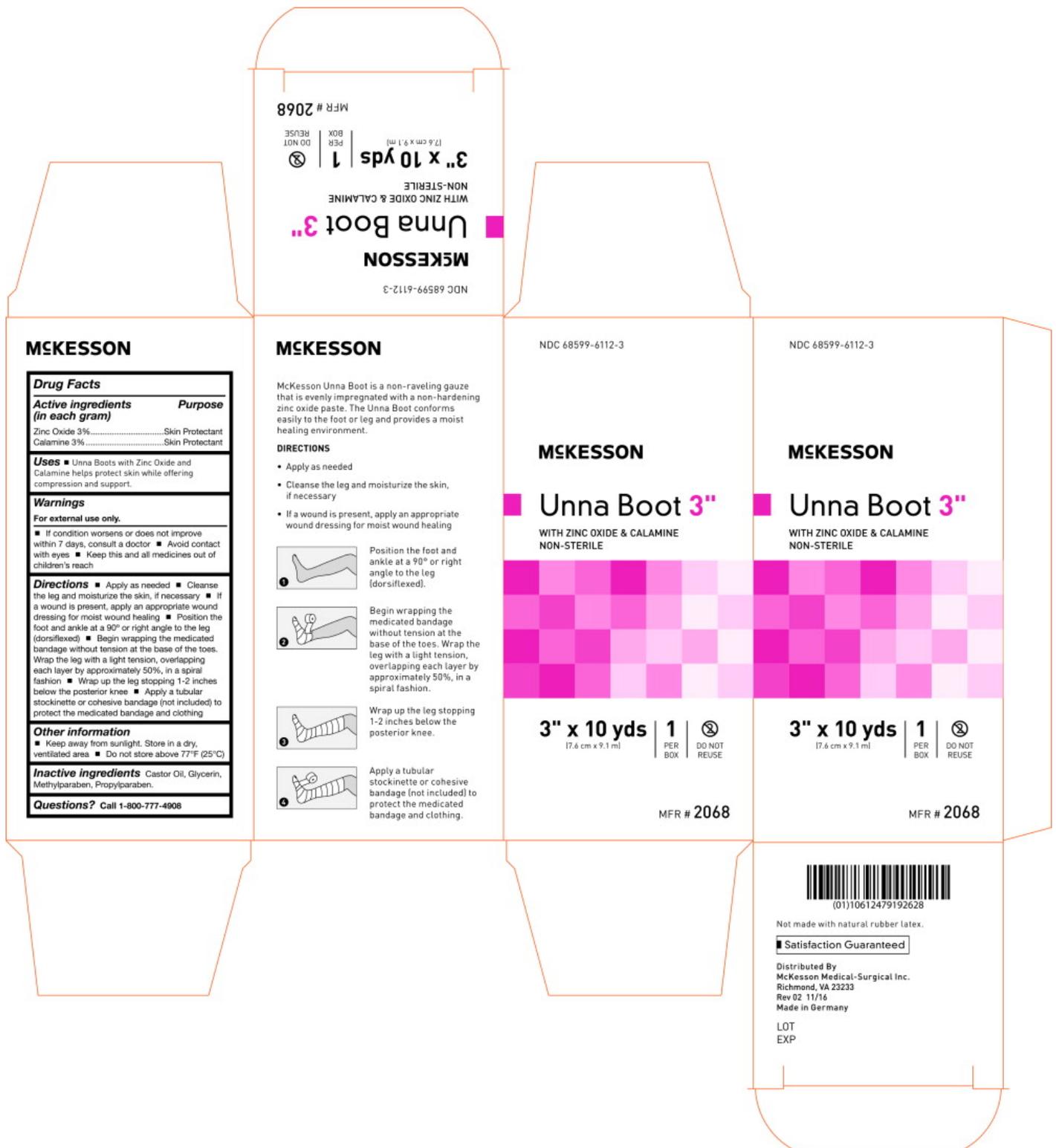

Principal Display Panel - Case Label

NDC 68599-6114-2

McKESSON

Unna Boots 4"
WITH ZINC OXIDE & CALAMINE | NON-STERILE

4" x 10 yds

(10.2 cm x 9.1 m)

1
PER BOX

12
BOXES PER CASE

DO NOT REUSE

Keep away from sunlight. Store in a dry, ventilated area. Do not store above 77°F (25°C). Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

MFR # **2069**

Distributed By

McKesson Medical-Surgical Inc.

Richmond, VA 23228

Rev. 01 04/14

Made in Germany

LOT

EXP

NDC 68599-6114-2

McKESSON

Unna Boots 4"

WITH ZINC OXIDE & CALAMINE | NON-STERILE

4" x 10 yds
(10.2 cm x 9.1 m)

1

PER BOX

12

BOXES PER CASE



DO NOT REUSE

Keep away from sunlight. Store in a dry, ventilated area. Do not store above 77°F (25°C). Not made with natural rubber latex.

Questions? Call 1-800-777-4908

■ Satisfaction Guaranteed

Distributed By
McKesson Medical-Surgical Inc.
Richmond, VA 23233
Rev 02 11/16
Made in Germany

MFR # **2069**



(01)20612479192663

LOT
EXP

Principal Display Panel

NDC 68599-6114-3

McKESSON


Unna Boot **4"**
WITH ZINC OXIDE & CALAMINE
NON-STERILE

4" x 10 yds
(10.2 cm a 9.1 m)

1
PER BOX

DO NOT REUSE

MFR # **2069**

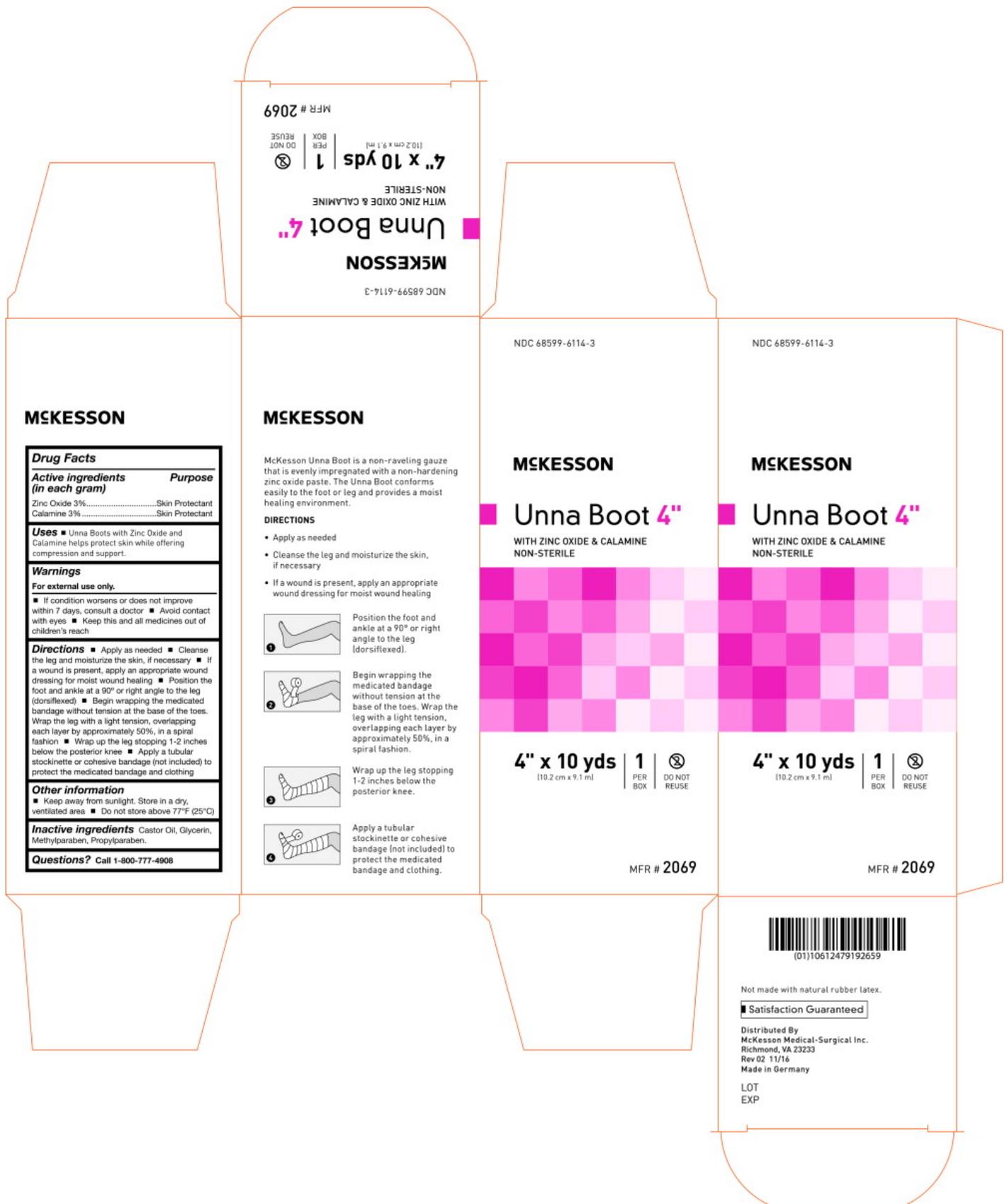

MFR # 2069

BOX
DO NOT
REUSE

1 4" x 10 yds
(10.2 cm x 9.1 m)

WITH ZINC OXIDE & CALAMINE
NON-STERILE

Unna Boot 4"

MCKESSON

NDC 68599-6114-3

NDC 68599-6114-3

NDC 68599-6114-3

MCKESSON

MCKESSON

MCKESSON

MCKESSON

Drug Facts	
Active ingredients (in each gram)	Purpose
Zinc Oxide 3%.....	Skin Protectant
Calamine 3%.....	Skin Protectant
Uses ■ Unna Boots with Zinc Oxide and Calamine helps protect skin while offering compression and support.	
Warnings	
For external use only.	
■ If condition worsens or does not improve within 7 days, consult a doctor ■ Avoid contact with eyes ■ Keep this and all medicines out of children's reach	
Directions ■ Apply as needed ■ Cleanse the leg and moisturize the skin, if necessary ■ If a wound is present, apply an appropriate wound dressing for moist wound healing ■ Position the foot and ankle at a 90° or right angle to the leg (dorsiflexed) ■ Begin wrapping the medicated bandage without tension at the base of the toes. Wrap the leg with a light tension, overlapping each layer by approximately 50%, in a spiral fashion ■ Wrap up the leg stopping 1-2 inches below the posterior knee ■ Apply a tubular stockinette or cohesive bandage (not included) to protect the medicated bandage and clothing	
Other information	
■ Keep away from sunlight. Store in a dry, ventilated area ■ Do not store above 77°F (25°C)	
Inactive ingredients Castor Oil, Glycerin, Methylparaben, Propylparaben.	
Questions? Call 1-800-777-4908	

McKesson Unna Boot is a non-raveling gauze that is evenly impregnated with a non-hardening zinc oxide paste. The Unna Boot conforms easily to the foot or leg and provides a moist healing environment.

DIRECTIONS

- Apply as needed
- Cleanse the leg and moisturize the skin, if necessary
- If a wound is present, apply an appropriate wound dressing for moist wound healing



Position the foot and ankle at a 90° or right angle to the leg (dorsiflexed).



Begin wrapping the medicated bandage without tension at the base of the toes. Wrap the leg with a light tension, overlapping each layer by approximately 50%, in a spiral fashion.



Wrap up the leg stopping 1-2 inches below the posterior knee.



Apply a tubular stockinette or cohesive bandage (not included) to protect the medicated bandage and clothing.

Unna Boot 4"

WITH ZINC OXIDE & CALAMINE
NON-STERILE

Unna Boot 4"

WITH ZINC OXIDE & CALAMINE
NON-STERILE

4" x 10 yds
(10.2 cm x 9.1 m)

1 PER BOX DO NOT REUSE

4" x 10 yds
(10.2 cm x 9.1 m)

1 PER BOX DO NOT REUSE

MFR # 2069

MFR # 2069



Not made with natural rubber latex.

Satisfaction Guaranteed

Distributed By
McKesson Medical-Surgical Inc.
Richmond, VA 23233
Rev 02 11/16
Made in Germany

LOT
EXP

MCKESSON UNNA BOOT WITH CALAMINE 4
zinc oxide and ferric oxide red dressing

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:68599-6114	
Route of Administration	TOPICAL			
Active Ingredient/Active Moiety				
Ingredient Name		Basis of Strength	Strength	
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC OXIDE - UNII:SOI2LOH54Z)		ZINC OXIDE	6.15 g in 205 g	
FERRIC OXIDE RED (UNII: 1K09F3G675) (FERRIC OXIDE RED - UNII:1K09F3G675)		FERRIC OXIDE RED	0.43 g in 205 g	
Inactive Ingredients				
Ingredient Name			Strength	
CASTOR OIL (UNII: D5340Y2I9G)				
GLYCERIN (UNII: PDC6A3C0OX)				
METHYLPARABEN (UNII: A2I8C7HI9T)				
PROPYLPARABEN (UNII: Z8IX2SC1OH)				
Packaging				
#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:68599-6114-2	12 in 1 CASE	07/01/2014	
1	NDC:68599-6114-3	205 g in 1 BOX; Type 4: Device Coated/Impregnated/Otherwise Combined with Drug		
Marketing Information				
Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date	
OTC monograph final	part347	07/01/2014		

MCKESSON UNNA BOOT WITH CALAMINE 3				
zinc oxide and ferric oxide red dressing				
Product Information				
Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:68599-6112	
Route of Administration	TOPICAL			
Active Ingredient/Active Moiety				
Ingredient Name		Basis of Strength	Strength	
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC OXIDE - UNII:SOI2LOH54Z)		ZINC OXIDE	4.95 g in 165 g	

FERRIC OXIDE RED (UNII: 1K09F3G675) (FERRIC OXIDE RED - UNII:1K09F3G675)

FERRIC OXIDE RED

0.35 g in 165 g

Inactive Ingredients

Ingredient Name	Strength
CASTOR OIL (UNII: D5340Y2I9G)	
GLYCERIN (UNII: PDC6A3C0OX)	
METHYLPARABEN (UNII: A2I8C7HI9T)	
PROPYLPARABEN (UNII: Z8IX2SC1OH)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:68599-6112-2	12 in 1 CASE	07/01/2014	
1	NDC:68599-6112-3	205 g in 1 BOX; Type 4: Device Coated/Impregnated/Otherwise Combined with Drug		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph final	part347	07/01/2014	

Labeler - McKesson Medical-Surgical (023904428)

Establishment

Name	Address	ID/FEI	Business Operations
Karl Otto Braun (KOB)		318330206	manufacture(68599-6112, 68599-6114)

Revised: 1/2022

McKesson Medical-Surgical