QCARE PETITE SIZE ORAL CLEANSING AND SUCTIONING SYSTEM- cetylpyridinium chloride

Sage Products, LLC

Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.

QCare Petite Size Oral Cleansing and Suctioning System

Drug Facts

Suction Swab, Petite Size, with Antiseptic Oral Rinse/Suction Toothbrush, Petite Size, with Antiseptic Oral Rinse

Active Ingredient:	Purpose
ANTISEPTIC ORAL RINSE: Cetylpyridinium chloride .05%	Antiseptic Rinse

Uses

Suction Swab with Antiseptic Oral Rinse

- Aids in the removal of secretions and debris and helps reduce the chance of infection in minor oral irritation.

Suction Toothbrush with Antiseptic Oral Rinse

- Aids in the removal of dental plaque, secretions, and debris; helps reduce the chance of infection in minor oral irritation.

Oropharyngeal Suction Catheter

- Aids in the removal of secretions from the oropharyngeal cavity only.

Warnings

Stop use and ask a doctor if:

- Sore mouth symptoms do not improve in 7 days.
- Swelling, rash or fever develops.
- Irritation, pain or redness persists or worsens.

Keep out of reach of children.

If more than used for antisepsis is accidentally swallowed, get medical help or contact a Poison Control Center right away.

DIRECTIONS

Suction Swab with Antiseptic Oral Rinse

- **Before opening**, turn package over, burst solution packet with thumbs.
- Peel lid to open.
- Remove Mouth Moisturizer and Applicator Swab.
- Attach Suction Swab to suction line.
- Clean teeth and oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.

- Discard Suction Swab. Reattach Covered Yankauer to suction line.
- Place Mouth Moisturizer on Applicator Swab.
- Apply as needed to lips and inside mouth.
- Use up to 4 times daily or as directed by a dentist or doctor.
- Children under 12 years of age: supervise use.
- Children under 3 years of age: consult a dentist or doctor.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Ensure foam is intact after use. If not, remove any particles from oral cavity.

Suction Toothbrush with Antiseptic Oral Rinse

- **Before opening**, turn package over, burst solution packet with thumbs.
- Peel lid to open.
- Remove Mouth Moisturizer and Applicator Swab.
- Attach Suction Toothbrush to suction line.
- Clean teeth and oral cavity for approximately one minute.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Toothbrush. Reattach Covered Yankauer to suction line.
- Place Mouth Moisturizer on Applicator Swab.
- Apply as needed to lips and inside mouth.
- Use up to 4 times daily or as directed by a dentist or doctor.
- Children under 12 years of age: supervise use.
- Children under 3 years of age: consult a dentist or doctor.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Ensure foam is intact after use. If not, remove any particles from oral cavity.

Oropharyngeal Suction Catheter

- Peel lid to open.
- Attach Suction Catheter to suction line.
- To suction, place thumb over port.
- To clear tubing, rinse with sterile saline or appropriate solution.
- Discard Suction Catheter. Reattach Covered Yankauer to suction line.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.

CAUTION:

- Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

Inactive Ingredients

Water, sorbitol, mentha piperita (peppermint) extract, potassium sorbate, polysorbate 80, polysorbate 20, citric acid, Blue 1 (CI 42090)

Questions?

Call toll-free 800-323-2220.

Mouth Moisturizer ingredients*

Water, cocos nucifera (coconut) oil, xylitol, flavor, cellulose gum, tocopheryl acetate (vitamin E), mentha veridis (spearmint) leaf oil, carbomer, polysorbate 20, polysorbate 80, potassium sorbate, cetylpyridinium chloride

* Manufactured for Sage Products LLC • Cary, IL

www.sageproducts.com/patents

NOT MADE WITH NATURAL RUBBER LATEX • FOR SINGLE USE ONLY • MADE IN USA

QCare Petite Size Oral Cleansing and Suctioning System

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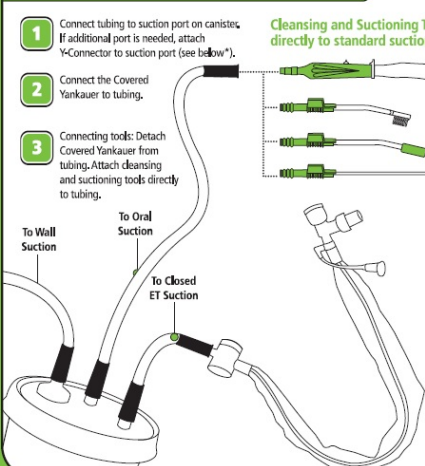
Q.Care®

Petite Size Oral Cleansing & Suctioning System

Setup Instructions: Multi-Port Canister

- 1 Connect tubing to suction port on canister. If additional port is needed, attach Y-Connector to suction port (see below*).
- 2 Connect the Covered Yankauer to tubing.
- 3 Connecting tools: Detach Covered Yankauer from tubing. Attach cleansing and suctioning tools directly to tubing.

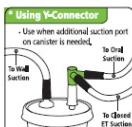
Cleansing and Suctioning Tools attach directly to standard suction lines.

Using the Covered Yankauer to aid with removal of secretions:

- Follow hospital protocol.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Retract sleeve until cap locks in place.
- To suction, move switch to ON. When finished, move switch to OFF to prevent loss of suction power.
- Rinse after use until tubing is clear.
- Between uses, pull sleeve over tip to cover.
- Discard Yankauer and Y-Connector (if used) after using for 24 hours, or according to hospital protocol.
- For single patient oral use.

Using Y-Connector

Use when additional suction port on canister is needed.

IMPORTANT: Both ports on Y-Connector must be utilized or a loss of suction power can occur.

Contents:

- Covered Yankauer and Y-Connector
- Petite Size Suction Toothbrushes, 7mL Antiseptic Oral Rinse, Applicator Swab and 2g Mouth Moisturizer
- Petite Size Untreated Suction Oral Swabs, 7mL Antiseptic Oral Rinse, Applicator Swab and 2g Mouth Moisturizer
- 2 French Oropharyngeal Suction Catheters

Caution:

- Refer to individual Suction Oral Swab, Suction Toothbrush and Oropharyngeal Suction Catheter packages for ingredients, indications, warnings and detailed instructions for use.
- Follow hospital protocol.
- For single patient oral use.
- Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
- Keep out of reach of children.

Ingredients:

Antiseptic Oral Rinse active ingredient: Cetylpyridinium chloride 0.05%
Mouth Moisturizer: Water, cocos nucifera (coconut) oil, xylitol, flavor, cellulose gum, tocopheryl acetate (vitamin E), mentha veridis (spearmint) leaf oil, carbomer, polysorbate 20, polysorbate 80, potassium sorbate, cetylpyridinium chloride. Non-sterile, not made with natural rubber latex.

Caution:
Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

Patents: www.sageproducts.com/patents

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SAGE®
 PRODUCT IS
 Sage Products LLC
 3909 Three Oaks Road
 Cary, Illinois 60013
 www.sageproducts.com
 800-323-2220 | 815-455-4700

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8 F Catheter / Sonde 8 F

3 53462-175-602

Oral Check | Scan bar code for compliance to protocol

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Drug Facts	Purpose
Active ingredient ANTISEPTIC ORAL RINSE: Cetylpyridinium chloride 0,05%,.....Antiseptic Rinse	
Uses • Aids in the removal of secretions and debris and helps reduce the chance of infection in minor oral irritation.	
Warnings Stop use and ask a doctor if: • Sore mouth symptoms do not improve in 7 days. • Swelling, rash or fever develops. • Irritation, pain or redness persists or worsens.	
Keep out of reach of children. If more than used for antiseptic is accidentally swallowed, get medical help or contact a Poison Control Center right away.	
Directions • Before opening, turn package over, burst solution packet with thumbs. • Peel lid to open. • Remove Mouth Moisturizer and Applicator Swab. • Attach Suction Swab to suction line. • Clean teeth and oral cavity for approximately one minute. • To suction, place thumb over port. • To clear tubing, rinse with sterile saline or appropriate solution. • Discard Suction Swab, Reattach Covered Yankauer to suction line. • Place Mouth Moisturizer on Applicator Swab. • Apply as needed to lips and inside mouth. • Use up to 4 times daily or as directed by a dentist or doctor. • Children under 12 years of age: supervise use. • Children under 3 years of age: consult a dentist or doctor. • Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands. • Ensure foam is intact after use. If not, remove any particles from oral cavity.	
Inactive ingredients Water, sorbitol, mentha piperita (peppermint) extract, potassium sorbate, polysorbate 80, polysorbate 20, citric acid, Blue 1 (CI 42090)	
Questions? Call toll-free 800-323-2220.	

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Suction Toothbrush

Petite size, with Antiseptic Oral Rinse

CONTENTS: 1 Suction Toothbrush, 0.25 fl. oz./7ml Antiseptic Oral Rinse, 0.07 oz./2g Mouth Moisturizer, 1 Applicator Swab

Drug Facts	Purpose
Active ingredient ANTISEPTIC ORAL RINSE: Cetylpyridinium chloride 0,05%,.....Antiseptic Rinse	
Uses • Aids in the removal of dental plaque, secretions and debris; helps reduce the chance of infection in minor oral irritation.	
Warnings Stop use and ask a doctor if: • Sore mouth symptoms do not improve in 7 days. • Swelling, rash or fever develops. • Irritation, pain or redness persists or worsens.	
Keep out of reach of children. If more than used for antiseptic is accidentally swallowed, get medical help or contact a Poison Control Center right away.	
Directions • Before opening, turn package over, burst solution packet with thumbs. • Peel lid to open. • Remove Mouth Moisturizer and Applicator Swab. • Attach Suction Toothbrush to suction line. • Clean teeth and oral cavity for approximately one minute. • To suction, place thumb over port. • To clear tubing, rinse with sterile saline or appropriate solution. • Discard Toothbrush, Reattach Covered Yankauer to suction line. • Place Mouth Moisturizer on Applicator Swab. • Apply as needed to lips and inside mouth. • Use up to 4 times daily or as directed by a dentist or doctor. • Children under 12 years of age: supervise use. • Children under 3 years of age: consult a dentist or doctor. • Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands. • Ensure foam is intact after use. If not, remove any particles from oral cavity.	
Inactive ingredients Water, sorbitol, mentha piperita (peppermint) extract, potassium sorbate, polysorbate 80, polysorbate 20, citric acid, Blue 1 (CI 42090)	
Questions? Call toll-free 800-323-2220.	

Mouth Moisturizer ingredients*: Water, cocos nucifera (coconut) oil, xylitol, flavor, cellulose gum, tocopheryl acetate (vitamin E), mentha viridis (spearmint) leaf oil, carboxymethylcellulose sodium salt, polysorbate 80, potassium sorbate, cetylpyridinium chloride
* Manufactured for Sage Products LLC • Cary, IL
Patents: www.sageproducts.com/patents

NOT MADE WITH NATURAL RUBBER LATEX FOR SINGLE USE ONLY • MADE IN U.S.A.

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Oropharyngeal Suction Catheter 8 F, Non-sterile

CONTENTS: 1 Suction Catheter

USES: Aids in the removal of secretions from the oropharyngeal cavity only.
DIRECTIONS:
• Peel lid to open.
• Attach Suction Catheter to suction line.
• To suction, place thumb over port.
• To clear tubing, rinse with sterile saline or appropriate solution.
• Discard Suction Catheter, Reattach Covered Yankauer to suction line.
• Use a bite block when performing oral care on patients with altered levels of consciousness or those who cannot comprehend commands.
CAUTION:
• Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

NOT MADE WITH NATURAL RUBBER LATEX FOR SINGLE USE ONLY • MADE IN U.S.A.

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QCARE PETITE SIZE ORAL CLEANSING AND SUCTIONING SYSTEM

cetylpyridinium chloride kit

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:53462-714
---------------------	----------------	---------------------------	---------------

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:53462-714-60	1 in 1 KIT; Type 1: Convenience Kit of Co-Package	04/02/2013	

Quantity of Parts

Part #	Package Quantity	Total Product Quantity
--------	------------------	------------------------

Part 1	6 PACKET	42 mL in 6
Part 2	6 PACKET	12 g in 6

Part 1 of 2

ANTISEPTIC ORAL RINSE

cetylpyridinium chloride mouthwash

Product Information

Item Code (Source)	NDC:53462-175
Route of Administration	BUCCAL

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
CETYLPIRIDINIUM CHLORIDE (UNII: D9OM4SK49P) (CETYLPIRIDINIUM - UNII:CUB7J10JV3)	CETYLPIRIDINIUM CHLORIDE	0.5 mg in 1 mL

Inactive Ingredients

Ingredient Name	Strength
SORBITOL (UNII: 506T60A25R)	
POTASSIUM SORBATE (UNII: 1VPU26JZZ4)	
POLYSORBATE 80 (UNII: 6OZP39ZG8H)	
POLYSORBATE 20 (UNII: 7T1F30V5YH)	
CITRIC ACID MONOHYDRATE (UNII: 2968PHW8QP)	
FD&C BLUE NO. 1 (UNII: H3R47K3TBD)	
WATER (UNII: 059QF0KO0R)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		6 in 1 KIT		
1	NDC:53462-175-60	7 mL in 1 PACKET; Type 0: Not a Combination Product		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	05/21/1998	

Part 2 of 2

MOUTH MOISTURIZER

other oral hygiene products emulsion

Product Information

Route of Administration BUCCAL

Other Ingredients

Ingredient Kind	Ingredient Name	Quantity
INGR	COCONUT OIL (UNII: Q9L0O73W7L)	
INGR	XYLITOL (UNII: VCQ006KQ1E)	
INGR	CARBOXYMETHYLCELLULOSE SODIUM (UNII: K679OBS311)	
INGR	.ALPHA.-TOCOPHEROL ACETATE (UNII: 9E8X80D2L0)	
INGR	WATER (UNII: 059QF0K00R)	
INGR	CARBOMER HOMO POLYMER TYPE B (ALLYL PENTAERYTHRITOL CROSSLINKED) (UNII: HHT01ZNK31)	
INGR	POLYSORBATE 20 (UNII: 7T1F30V5YH)	
INGR	POLYSORBATE 80 (UNII: 6OZP39ZG8H)	
INGR	POTASSIUM SORBATE (UNII: 1VPU26JZZ4)	
INGR	CETYLPYRIDINIUM CHLORIDE (UNII: D9OM4SK49P)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		6 in 1 KIT		
1		2 g in 1 PACKET; Type 0: Not a Combination Product		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
Cosmetic			

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph not final	part356	04/02/2013	

Labeler - Sage Products, LLC (054326178)