

**ACESO AG SILVER SILICONE FOAM WOUND DRESSING- dressing, wound, drug  
NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY**

-----  
**ACESO Ag, Silver Silicone Foam Wound Dressing**

-----  
**ACESO Ag, Silicone Silver Foam Dressing, Rx Only** ACESO Ag, Silicone Silver Foam Dressing consists of 0.25-0.35mg/cm<sup>2</sup> silver ions, and it has been shown that antibacterial effectiveness within the dressing for up to 7 days, as demonstrated in vitro.

**Indications**

For prescription use only.

ACESO Ag, Silicone Silver Foam Dressing is indicated for the treatment of exuding wounds, such as leg and foot ulcers, pressure ulcers, traumatic and surgical wounds, superficial and partial thickness burns.

**DESCRIPTION**

ACESO Ag, is an absorbent Silicone Silver Foam Dressing used for the management of exudative wounds. It provides a physiologically favorable moist wound environment for wound healing. ACESO Ag absorbs excess wound exudate. ACESO Ag uses silver as a preservative to control bacterial growth within the foam dressing.

ACESO Ag, Silicone Silver Foam Dressing provides a physiologically favorable environment that encourages wound healing and conforms to the wound site. It is easy to use, biocompatible and biodegradable. ACESO Ag, supports natural autolysis by rehydrating and softening necrotic tissue and eschar thereby encouraging autolytic debridement. The silver preservative in the dressing controls microbial growth for up to 7 days within the dressing, as demonstrated in vitro.

**Product Administration**

1. Clean the wound with sterile water or normal saline and dry the surrounding skin gently.
2. Remove the release films to cover the wound according to the wound area. The dressing should overlap the dry surrounding skin by at least 1-2 cm.
3. The frequency of replacing depends on patient's conditions and the exudates situation, such as every 24 hours or longer, but cannot exceed more than 7 days.

Note: Dressing may be moistened with sterile water or normal saline to ease removal.

**Warnings**

1. ACESO Silicone Ag Foam Dressing is a single use. Do not reuse.
2. ACESO Silicone Ag Foam Dressing is sterile. Do not use it if the pouch is damaged or opened prior to use.
3. Do not suggest using this product on dry wounds.
4. Do not use ACESO Silicone Ag Foam Dressing together with oxidant, such as hypochlorite solutions or hydrogen peroxide.
5. ACESO Silicone Ag Foam Dressing may cause transient discoloration of the wound bed and surrounding skin. Frequent or prolonged use of this product may result in permanent discoloration of skin.

6. ACESO Silicone Ag Foam Dressing does not replace the need for systemic therapy or other adequate infection treatment in the event of clinical infection.
7. Do not use ACESO Silicone Ag Foam Dressing during radiation treatment or examinations, such as X-ray, magnetic resonance imaging [MRI] ultrasound or diathermy.
8. Avoid contact with electrodes or conductive gels during electronic measurements, such as electrocardiograms [ECG], and electroencephalograms [EEG].
9. ACESO Silicone Ag Foam Dressing is only used for external use.

## **CONTRAINDICATIONS**

Contraindicated for individuals with a known sensitivity to silver and any other contents of the dressing.

## **Cautions**

Federal law restricts this device to sale by or on the order of a physician. (Rx Only)  
The total maximum amount of silver is 0.35mg/cm<sup>2</sup> in wound pad.

## **How Supplied**

ACESO Ag, Silver Silicone Foam Wound Dressings are packaged in 4 sterile bandages per pack.  
NDC 72678-003-01.

## **Storage and Disposal**

1. Please store the product in cool and dry place, and avoid light.
  2. Disposal should be handled according to local environmental procedures.
- SHELF LIFE: 3 years from date of manufacture.

## **Manufactured for:**

National Bio Green Sciences LLC, Branchburg, NJ  
Questions: National Bio Green Sciences, LLC  
Call: 1-908-255-4133 or  
Email: [info@nbgusa.com](mailto:info@nbgusa.com)

前短边3mm

10mm

160 mm





**ACESO**  
Silicone Silver Foam Dressing  
Bordered



1 Remove dressing from package. Remove one piece of the release liner.



2 Place dressing on the wound and remove the other release liner.



3 Gently secure the dressing to the skin.

STERILE EO

Caution Do not re-use Do not use if package is damaged

Keep away from sunlight Keep dry Temperature limit 40°C

**Storage:** Store in a cool, ventilated and dry place, avoid light.

**Caution:** The total maximum amount of silver is 0.35mg/cm<sup>2</sup> in wound pad. Please see the package insert for complete instructions for use.

Manufactured for: National Bio Green Sciences LLC  
6b Culnen Drive Branchburg NJ 08876

**Rx Only**

**NDC:72678-003-01**

4"x4"  
(10cm x 10cm)


LOT:

EXP:

8mm

145 mm

V6  
Revised 4/29/19  
Loukas Renieris

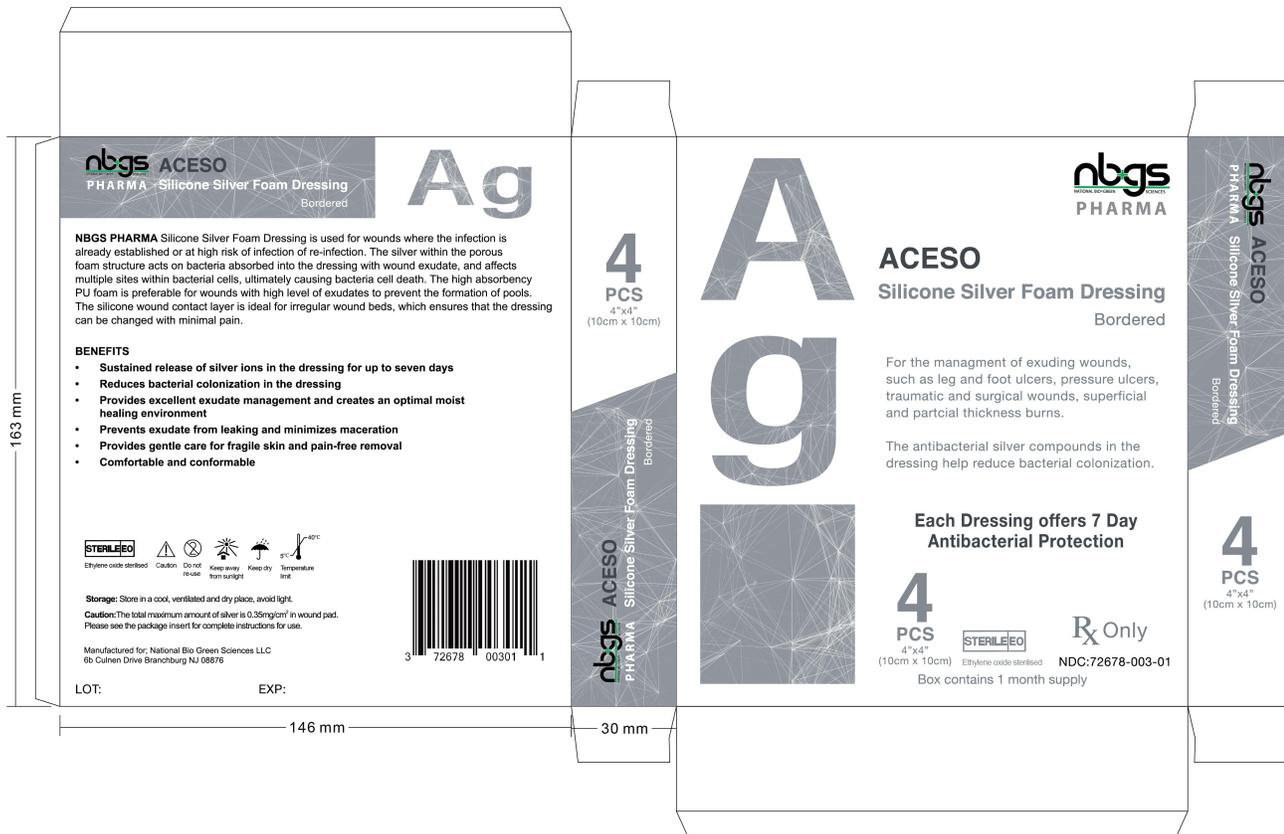

V6  
Revised 4/29/19  
Loukas Renieris

## ACESO AG SILVER SILICONE FOAM WOUND DRESSING

dressing, wound, drug

### Product Information

<b>Product Type</b>	MEDICAL DEVICE	<b>Item Code (Source)</b>	NHRIC:72678-003
---------------------	----------------	---------------------------	-----------------

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:72678-003-01	4 in 1 CARTON; Type 0: Not a Combination Product		

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
premarket notification	K170077	07/15/2019	

**Labeler** - NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY (967054623)

Revised: 8/2019

NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY